

BRUNO

Space-Saver™



Model
ASL-325

Only BRUNO tests lift designs to three times rated-weight capacity!



- ◆ The Bruno Model ASL-325 Space-Saver™ Vehicle Lift is an exceptionally versatile take-apart lift designed to help store your scooter in vehicles with smaller trunks. The Space-Saver solves the problem of storing a scooter in smaller vehicles with limited trunk space.
- ◆ The Space-Saver is designed to fit closer to the interior trunk wall, providing a 'tighter arc' when manually swinging the scooter in/out of the trunk. This allows scooters to be stored in spaces that won't work with more conventional lift designs, and the rotation of the lift arm makes it possible to raise/lower the scooter at the rear or side of the vehicle.
- ◆ The Space-Saver is fully adjustable for height and length and raises scooters weighing up to 200 lbs. The lift tower base can be easily adjusted to fit nearly any car's trunk, ensuring a useful, reliable installation. The Space-Saver is lightweight, and assembly/disassembly can be completed in three simple steps, quickly and easily.
- ◆ The Space-Saver can be stored anywhere – in the trunk or inside the vehicle itself. And when an additional tower base installation kit is ordered, the Space-Saver can be used in more than one vehicle. Contact Bruno's Technical Application Department for details.

OPTIONAL BRUNO POW'RPAC



- ◆ Easy to use!
- ◆ Convenient!
- ◆ Works in most minivans, SUVs, station wagons & sedans!

The complete, independent power supply for your automotive lift!

Pow'rPac easily installs in your vehicle – just plug into the vehicle's power outlet!

- ◆ **NEW!** The Bruno Exterior Hitch Mount is now available for the Space-Saver Lift. It's ideal for nearly any Coupe, Sedan, Hatchback, Wagon and Sport Utility Vehicle on the market such as the Ford Focus ZX3 and ZX5, Honda CRV, Toyota RAV4... and more!
- ◆ **EXCLUSIVE!** Bruno's R³ Lift Belt is 50% stronger! Count on Bruno's R³ Lift Belt for ease of use, longer life and safer operation – *another Industry First!*
- ◆ Expert installation and complete, after-sale service are provided by Bruno's nationwide network of dealers.
- ◆ Only Bruno has over 25 different interior/exterior scooter, wheelchair and power chair lifts for nearly any sedan, hatchback, station wagon, convertible, minivan, sport utility vehicle (SUV), full-size van, pickup truck and the new cross-over vehicles.

◆ All Bruno lifts are eligible for reimbursement under the terms of Mobility Programs offered by GM, Saturn, DaimlerChrysler/Dodge, Ford, Toyota, Lexus, Volvo and Volkswagen.



Using the *Bruno Space Saver*[®] is easy!

1



Once the scooter is positioned at the rear of your car, remove the seat and hook the *lift claw* to the T-bar docking device.

2



To lift the scooter, push the **UP** button until the wheels clear the trunk opening.

3



Once the scooter is at the proper height, manually rotate the scooter over the trunk opening.

4



With the scooter in position, push the **DOWN** button until the scooter rests on the trunk floor.

5



Disconnect the *lift claw* from the scooter's T-bar docking device and disassemble the lift. The heaviest piece weighs only 13 lbs. (6kg).

6



The three pieces of the Space-Saver Lift assembly can be stored with the scooter or inside your vehicle.

Bruno lifts are eligible for reimbursement under the terms of Mobility Programs offered by GM, Saturn, DaimlerChrysler/Dodge, Ford, Toyota, Lexus and Volkswagen.

Represented By:



The Lift weight rating is the MAXIMUM load value. Factors inherent to specific applications including, but not limited to, vehicle structure and condition, operating environment, and customer ability may reduce the effective weight rating.



Bruno Independent Living Aids, Inc., 1780 Executive Drive
P.O. Box 84, Oconomowoc, WI 53066
(262)567-4990 ♦ FAX (262) 953-5501
www.bruno.com